



May 16, 2011

Dear Valued Customer:

There is an update to the Gulf Medical Fiberoptics Surgical Headlight User Guide. It has come to our attention that cleaning the exterior surfaces of the headlight, headgear, and fiber optic cable with 70% isopropyl alcohol has not been validated.

This update affects the following Gulf Medical Fiberoptics Surgical Headlight part numbers:

- GMF-H001
- GMF-H100
- GMF-H101
- GMF-H102
- GMF-H103
- GMF-H104
- GMF-H105
- GMF-H106

At this time, please discontinue the use of 70% isopropyl alcohol to clean the surfaces of this device. We will notify you through written communication of any future updates. Enclosed, please find an updated Surgical Headlight User Guide. Please notify all end users of this product. Please update any inventoried product with the updated user guide and discard/shred/recycle the prior version.

If you have any questions regarding this update, please contact our Customer Service department manager Jacqueline Armand at: +1 813.855.6618 x.427 or jacqueline@gulffiberoptics.com.

Sincerely,

A handwritten signature in black ink, appearing to read "Craig S. Vogeley".

Craig S. Vogeley
Vice President, Operations
Gulf Medical Fiberoptics, Inc.